

Choose from Classic or Premium Upholstery



Classic Upholstery
Model 3503-500-300



Premium Upholstery
Model 3503-500-305

Two Options to Choose

Every FusionONE and FusionONE+ model is available with Classic or Premium upholstery. Experience enhanced comfort, elegance, and softness in 16 stylish colors. The Premium upholstery model will elevate the aesthetics and comfort of any space.

Our upholstery with PreFixx® Protective Finish is available in 16 colors

Both options feature seamless durability & include PreFixx® Protective Finish designed for easy cleaning. This helps to reduce the risk of infection in the outpatient setting, especially in vulnerable patient populations, while maintaining the aesthetics of the exam room.

Classic Upholstery

Models

- 3001
- 3002
- 3002-500-300
- 3003
- 3003-500-300
- 3502
- 3502-500-300
- 3503
- 3503-500-300

Premium Upholstery

Models

- 3001-500-105
- 3002-500-105
- 3002-500-305
- 3003-500-105
- 3003-500-305
- 3502-500-105
- 3502-500-305
- 3503-500-105
- 3503-500-305

Upholstery and Colors



Need a tip card sent to you? Call us at 1-800-638-5322

Exam Table Upholstery with PreFixx® Protective Finish

Helping to enhance infection prevention initiatives


In laboratory testing, upholstery protected with PreFixx® finish was treated with the following disinfectants with little to no discoloration or damage to the upholstery. This testing may not reflect actual results in the field.*

Recommended Disinfectants**

- › Ecolab Quaternary Disinfectant Wipes
- › Agar™ Powerquat
- › Asepticare™
- › Asepticare™ TB-II
- › AVISTAT-D™ Ready To Use Spray Disinfectant Cleaner
- › Biotrol BirexSE®
- › Bleach 1:5 (20% bleach)
- › Bleach 1:9 (10% bleach)
- › Bleach-Rite® Disinfecting Spray
- › CaviCide™
- › CaviCide1™
- › CaviCide™ AF
- › Clorox® Broad Spectrum Quaternary Disinfectant Cleaner
- › Clorox® Healthcare Bleach Germicidal Cleaner
- › Clorox® Healthcare Bleach Germicidal Wipes
- › Clorox® Healthcare EZ-KILL® Wipes
- › Clorox® Healthcare VersaSure® Cleaner Disinfection Wipes
- › Clorox® Hydrogen Peroxide Cleaner Disinfectant Spray
- › Clorox® Hydrogen Peroxide Cleaner Disinfectant Wipes
- › Diversey™ Accel® INTERvention® Wipes
- › Diversey™ Avert® Sporicidal Disinfectant Cleaner
- › ERC Performance Wipes
- › McKesson Disposable Germicidal Surface Wipes
- › McKesson Pro-Tech RTU Disinfectant Cleaner
- › OPTIM® 1 Wipes
- › OPTIM® 33TB
- › Oxivir® 1 RTU
- › Oxivir® Five 16
- › Oxivir® TB
- › OxyCide™ Daily Disinfectant Cleaner
- › PDI Sani-Cloth® AF3 Germicidal Disposable Wipes
- › PDI Sani-Cloth® HB
- › PDI Sani-Cloth® Plus
- › PDI Sani-Cloth® Prime Germicidal Disposable Wipe
- › PDI Super Sani-Cloth®
- › PDI Super Sani-Cloth® Bleach Germicidal Disposable Wipes
- › Precise QTB Spray
- › PROCHEM® Oxy Plus
- › Purell® Healthcare Surface Disinfectant
- › Purell® Multi-Surface Disinfectant
- › Sani Professional® Cleaning + Degreasing Multi-Surface Wipes
- › Sani Professional® Multi-Surface Cleaning Wipes
- › Sani Professional® No-Rinse Sanitizing Multi-Surface Wipes
- › SaniZide® Plus
- › STERI-7 XTRA CONCENTRATE (recommended dilution ratio 1:10)
- › STERI-7 XTRA WIPES
- › Vert-2-Go ED
- › Virox AHP 5
- › Virox PREempt™ RTU
- › Virex® II 256
- › Virex® Plus One-Step Disinfectant Cleaner & Deodorant
- › Wayne® Concept 256N
- › Wex-Cide 128

**All disinfectants and cleaning agents contain chemicals that degrade coated fabric upholstery to some extent. To promote a long product life, it is recommended that the PreFixx cleaning and maintenance protocol be employed regularly.*

***Recommended disinfectants are only recommended when following the disinfectant manufacturer's instructions for application, and inclusion in this document does not imply "fit for use." Customers should first determine if products are appropriate for use on their surfaces.*

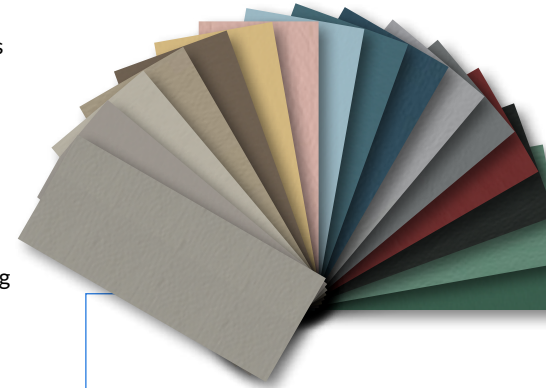

Care & Cleaning

Use one of the following cleaners with a soft cloth or damp sponge. Rinse area with fresh water then dry with a clean, lint-free cloth.

Primary Recommended Cleaner

- › Formula 409 All-Purpose spray cleaner
- › Fantastik spray cleaner

Secondary Recommended Cleaners

- › Lysol Clean and Fresh Multi-Surface Cleaner (Reckitt Benckiser)
- › Lestoil Heavy Duty Cleaner (Clorox)
- › Mr. Clean / Flash Clean and Shine (Procter and Gamble)
- › Eco Touch All Purpose Premium Care

For more difficult stains, contact UMF Medical Customer Service.



ALL UMF MEDICAL PRODUCTS ARE MADE IN THE USA

Ph: +1 814.266.8726 | Fax: +1 814.266.1870
1316 Eisenhower Blvd., Johnstown, PA 15904 USA

UMFmedical.com

©2023 United Metal Fabricators Inc. dba UMF Medical 09.23 All Rights Reserved.